

Erectease™

The Vacuum Pump for a Simple Fast Erection. Including:

Erectassist™

The simple way to maintain an Erection



Features

- Outstanding value for money
- Quick and easy to use – 10 second preparation
- Sustained erection for up to 30 minutes
- Ideal exercise device for penile enlargement as an adjunct to other therapies, five minutes a day for three weeks noticeably increases size and rigidity
- Safe and reliable for long-term users
- Low incidence of side-effects
- Complete and simple user control
- A compact, silent device in a discreet carry case
- A suitable option for a wide range of patients including those who have failed other therapies



mediwatch

Providing Complete Urological Diagnostic Services

Erectease™

The ERECTEASE is an external vacuum system – a mechanical, non-surgical method of filling the penis with blood and simulating a natural erection.



The simple way to maintain an Erection

Erectassist includes everything you would get with the Erectease system excluding the vacuum pump. It provides a simple and convenient way to keep blood within the penis once a natural erection has been attained.

One of the two sizes of transfer collar is fitted to the base of the pump. The loading cone (with 1 or both parts use to fit the selected collar) is used to stretch the correct size 'retention ring' over the transfer collar. This collar is placed over the penis and the retention ring is slid off and onto the base of the penis. This comfortably restricts the blood flow out of the penis and thereby maintains the erection.

Erectassist™

A normal erection involves an eight-fold increase in the amount of blood flowing into the penis. Erectile dysfunction or impotence can be caused by:

- Drug or alcohol abuse
- Diabetes
- Vascular disease
- Brain injury
- Spinal column injury (where sensation has not been lost)
- Strokes
- Invalidity due to paralysis
- MS
- Post surgical procedures
- Medication
- Psychogenic factors
- Tight foreskin
- Peyronies disease
- Endocrinological disorders

A Simple and Fast Erection

Erectease is a mechanical, non-surgical method of filling the penis with blood and simulating a natural erection. Patients select one of three soft suitably sized retention rings and attach it to the vacuum chamber. The vacuum chamber is placed over the penis and pumped, which quickly draws blood into the penis. During this process pressure is controlled using the finger valve.

When fully erect, the ring slides onto the penis base to maintain the appropriate BP, and then the vacuum chamber is removed. The patient proceeds with intercourse before reducing BP by pulling the ring handles outwards to enable the penis to go back to a flaccid state before removing the ring completely.

Contents:

- A – Pump
- B – Loading cones
- C – Sealing and transfer Collar
- D – Lubricant
- E – Retention rings – 3 sizes; 25mm, 32mm & 40mm diameter
- F – Finger Valve

The kit of parts fit into a discreet carry bag

Accessories

Description	Part Number
Erectease (With vacuum pump, carry case, loading cones, transfer collars, 3 assorted size retention rings & lubricant gel)	S0001
Erectassist (With loading cones, transfer collars, 3 assorted size retention rings & lubricant gel)	S0007
Extra set of 3 mixed size silicone tension rings.	S0101
Extra set of 3 small silicone tension rings.	S0101A
Extra set of 3 medium silicone tension rings.	S0101B
Extra set of 3 large silicone tension rings.	S0101C
Extra Lubricating gel for Erectease	S0106

CE 0843